

EYE WASH STATION 2 X 500ML BOTTLES OF STERILE SOLUTION

PRODUCT CODE: ASE000-000-000

2 BOTTLES OF EMERGENCY EYEWASH; A STERILE SOLUTION OF SODIUM 0.9 w/v (NORMAL SALINE) IN A CONVENIENT COVERED PLASTIC STATION

TECHNICAL SPECIFICATION

EYE & FACE PROTECTION

INSTRUCTIONS FOR USE

Twist off cap bottle, then irrigate the eye

Allow the fluid to flow freely across the eye

Once a bottle has been opened the remaining contents must be disposed of

PRECAUTIONS

Ensure that the solution is clear and seal intact before use. If not do not use

Check for leaks before use by squeezing the bottle

Dispose of any surplus


Keep out of the reach of children

ANY INCIDENT CONCERNING THE EYES MUST BE TAKEN SERIOUSLY. REQUEST MEDICAL ADVICE AS SOON AS POSSIBLE AFTER USING THE EYEWASH

DIMENSIONS OF OUTER PACKAGING

Quantity: 20

Weight: 11.5Kg (Approx.)

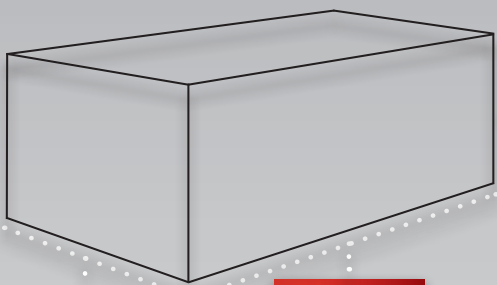

HEIGHT:
390mm

WIDTH:
465mm

LENGTH:
530mm

DISPOSAL INSTRUCTIONS

Empty the contents and incinerate at 400°C (minimum), or dispose by landfill at an approved site



SOLUTION:
Sodium Chloride 0.9 w/v,
in purified water PhEur
(Normal Saline)



WARNINGS

NOT FOR INJECTION

Not to be taken internally

SIDE EFFECTS

Oral ingestion of substantial quantities may require injection of diuretic to remove excess sodium

STORAGE

Recommended storage conditions - should be stored between 5°C and 25°C

Shelf Life is 3 years

Can be wall mounted or free standing and is dustproof and waterproof when covered

CONFORMITY

Medical Device, Class 1 Sterile

Conforms to HSE regulations

EC Council Directive 93/42/EEC

Meets the requirements of the Health and Safety Act 1974, requiring the provision of eyewash stations within the workplace



MANUFACTURING FOR SAFETY
WWW.JSP.CO.UK

UKSALES@JSP.CO.UK
TEL: +44 (0)1993 826050

